



### Instructions

- 1 Connect the patient camera to your video conference equipment.
- 2 Press the **Power** button to turn the camera on and press **Skip** when prompted for a log in.
- 3 Press the **Aux** button on the Tandberg remote control to display the patient camera image to the far end site.
- 4 Point the camera at the subject or image and press the **Zoom +/-** buttons on back of camera. The camera will automatically refocus.
- 5 If a still image is required, zoom in and out until you have the appropriate image in view then press and hold the **Image Capture** button on top of camera to freeze the image. Press it again to unfreeze the image and return to normal viewing.
- 6 Once completed, press the **Main Cam** button on the Tandberg remote control to return to displaying the main camera image to the far end site.
- 7 Turn off the power by pressing and holding the **Power** button.

### Additional Instructions

Use the **Light** for oral viewing during ENT and Anesthetic assessments.

You may want to keep the camera plugged in during the session if the battery does not hold the charge long enough. **Note: Plug the camera in to recharge the batteries when not in use.**