



Instructions

- 1 Connect the patient camera to your video conference equipment.
- 2 Press the **Power** button to turn the camera on and press **Skip** when prompted for a log in.
- 3 Press the **Aux** button on the Tandberg remote control to display the patient camera image to the far end site.
- 4 Point the camera at the subject or image and press the **Zoom +/-** buttons on back of camera. The camera will automatically refocus.
- 5 If a still image is required, zoom in and out until you have the appropriate image in view then press and hold the **Image Capture** button on top of camera to freeze the image. Press it again to unfreeze the image and return to normal viewing.
- 6 Once completed, press the **Main Cam** button on the Tandberg remote control to return to displaying the main camera image to the far end site.
- 7 Turn off the power by pressing and holding the **Power** button.

Additional Instructions

Use the **Light** for oral viewing during ENT and Anesthetic assessments.

You may want to keep the camera plugged in during the session if the battery does not hold the charge long enough.

Note: Plug the camera in to recharge the batteries when not in use.